


Sizing Guide for Millennial Advantage Crutches

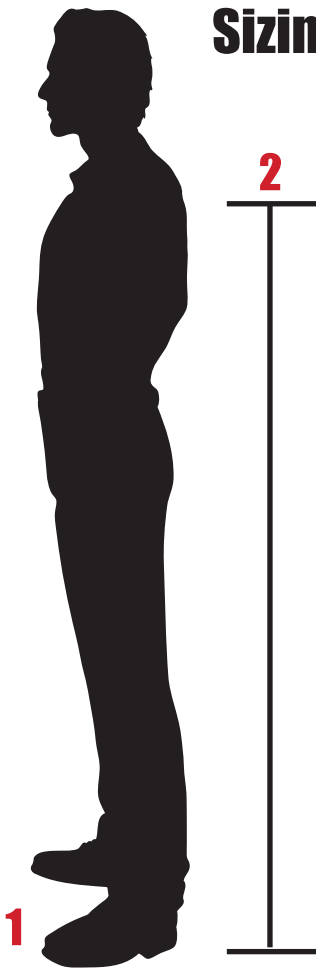

INSTRUCTIONS

- 1** Wear the shoes you will most commonly wear during crutch use.
- 2** Standing upright with good posture, measure from the floor to your *underarm.
- 3** Using this measurement, look at the range of handle heights in the short and tall Advantage crutches listed below.

Wherever your measurement falls is the crutch size you should use.

- 3** **Millennial Advantage Short** (MWD4000)
Floor to Underarm Cradle - 42.5" to 50.5" with 9 positions
- Millennial Advantage Tall** (MWD4500)
Floor to Underarm Cradle - 50" to 62.5" with 13 positions

*Millennial Advantage crutches are usually the most comfortable when placed 1" below the underarm, creating a slight bend in the elbow and allowing the user to stand up straight.